

# Renew®

PROFESSIONAL STRENGTH

DENTURE CLEANER  
PROTEZA CLEANER

RENEW®  
CANADA

200 g (7 oz)

**REMOVES:** Tobacco, coffee, tea, iron, calcium, calculus, plaque, tartar. Use 1 teaspoon of powder per cup of warm water. Stir solution thoroughly in a denture bath. **IMMERSE DENTAL APPLIANCE FOR 20 MINUTES.** Extremely stained dentures or dentures with heavy build-up of calculus and tartar may require overnight soaking. Do not soak metal partial dentures for more than 20 minutes. **IMPORTANT:** Dentures with soft liner should not be soaked for more than 20 minutes per soaking. **SAFETY CAUTION:** Renew is not to be taken internally. If swallowed do not induce vomiting, give large quantities of water. Call physician immediately. In case of contact with eyes or mouth flush with water. **ALLERGIC REACTION:** As with any denture cleaner an allergic reaction can occur. If an allergic reaction occurs, immediately discontinue use. **RINSE THE DENTAL APPLIANCE** thoroughly before using. **KEEP OUT OF REACH OF CHILDREN.** **DO NOT** use to clean jewellery. **DO NOT** mix with other chemicals. **AVOID** spilling on clothing, fabrics and countertops.

**ODSTRANI:** Tobak, čaj, železo, kalcij, kamen, obloge. Uporabite 1 žličko prahu na skodelico tople vode. Temeljito premešajte v posodi za zobno protezo. **POTOPITE ZOBNO PROTEZO ZA 20 MINUT.** Zelo umazane proteze ali proteze z veliko kamna in oblog bodo morda potrebovale čez-nočno namakanje. Ne namakajte kovinskih delnih protez dlje od 20 minut. **POMEMBNO:** Protez z mehko oblogo ne smete namakati dlje od 20 minut naenkrat. **VARNOSTNO OPOZORILO:** Renew se ne jemlje interno. Če je pogoltnjen, ne silite bruhanja, dajte velike količine vode. Pokličite zdravnika takoj. Pri stiku z očmi ali ustmi - sperite z vodo. **ALERGIČNA REAKCIJA:** Kot pri vsakemu čistilcu protez, se lahko pojavi alergična reakcija. Če se pojavi alergična reakcija, nemudoma prenehajte z uporabo. **IZPERITE ZOBNI APARAT** temeljito pred uporabo. **HRANITE IZVEN DOSEGA OTROK.** NE uporabljajte za čiščenje nakita. NE mešajte z drugimi kemikalijami. **IZOGIBAJTE** se razlivanju po oblačilih, tkanini in pulu.



Made in Canada/Narejeno v Kanadi

Manufactured by/Proizvajalec



**MID-CONTINENTAL**  
DENTAL SUPPLY CO. LTD.

Headingley, MB R4J 1A4 • Ph: 1-800-882-7341  
www.mid-continental.com



6 26103 00001 5